

Emerald Rigid Intubation Stylet

Pack of 25 Pcs

- Overall Length: 360mm (14.17")
- Working Length: 266mm (10.47")
- Ball Size: 5mm
- Clinically clean, packed in clean room and ETO sterilized
- Provides maneuverability for placement of an endotracheal tube
- Easy to high-level disinfect sterilized by low temperature ethylene oxide gas.
- Designed for use with endotracheal tubes.
- Easy to use, learn, and teach
- Load the endotracheal tube onto the stylet, ensure that the distal end of stylet does not extend beyond the distal end of the ET tube.
- Position the ET tube at the opening of the vocal cords. Do not advance the stylet past the vocal cords.
- When the ET tube is engaged in the glottic openings, extract the stylet 5 cm by use of the thumb pushing up on the thumb tab of the stylet. The partial removal of the stylet from ET tube softens the tip, allowing the tube to pass through the vocal cords.
- Position the ET tube as per standard practice.
- Remove the stylet from ET tube.
- Unit of Sale – 25 Pcs / Box

